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CHECK PRIOR TO USE: Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, DO NOT USE it. Get a new one.

A FITTING INSTRUCTIONS: ① Hold the respirator in hand with the nosepiece at your fingertips. Allow headbands to hang freely behind hand. ② Place the respirator firmly against your face with the nosepiece on the bridge of your nose. Stretch and pull the lower headband over the head and position below your ears. ③ Stretch and pull the top headband on the back of your head above your ears. ④ Press soft nosepiece to conform snugly around the nose. ⑤ (a) To test for respirator without exhalation valve: Cup both hands over the respirator and around the nose. ⑥ (b) To test for respirator with exhalation valve: Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit. ⑦ *Note: Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observation of these instructions is an important step in safe respirator use.

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CHECK PRIOR TO USE: Inspect respirator, including any component parts, for holes, tears or damage prior to use. Should any damage be identified, DO NOT USE it. Get a new one.

B MEANING OF PICTOGRAMS/SYMBOLS: ① See user information. ② Temperature range of storage condition. ③ Maximum relative humidity of storage condition. ④ Expiry date: printed on the product. ⑤ NR: denotes respirator is not suitable for use until after the expiry date. ⑥ (a) To test for respirator without exhalation valve: Cup both hands over the respirator and around the nose. ⑦ (b) To test for respirator with exhalation valve: Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit. ⑧ *Note: Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observation of these instructions is an important step in safe respirator use.

MANUFACTURED/IMPORTED BY: Refer to packaging for contact details.

B MEANING OF PICTOGRAMS/SYMBOLS: ① See user information. ② Temperature range of storage condition. ③ Maximum relative humidity of storage condition. ④ Expiry date: printed on the product. ⑤ NR: denotes respirator is not suitable for use until after the expiry date. ⑥ (a) To test for respirator without exhalation valve: Cup both hands over the respirator and around the nose. ⑦ (b) To test for respirator with exhalation valve: Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the headband for better fit. ⑧ *Note: Change respirator immediately if breathing becomes difficult or respirator becomes damaged or distorted, or a proper face fit can not be maintained. Careful observation of these instructions is an important step in safe respirator use.

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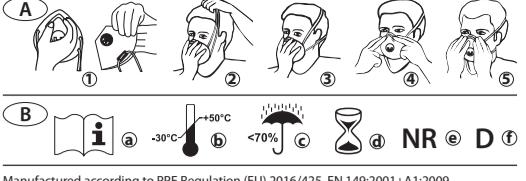
B MEANING OF PICTOGRAMS/SYMBOLS: ① See user information. ② Temperature range of storage condition. ③ Maximum relative humidity of storage condition. ④ Expiry date: printed on the product. ⑤ NR: denotes respirator is not suitable for use until after the expiry date. ⑥ (a) To test for respirator without exhalation valve: Cup both hands over the respirator and around the nose. ⑦ (b) To test for respirator with exhalation valve: Cup both hands over the respirator and inhale sharply. A negative pressure should be felt inside respirator.

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